Avoimella mansetilla varustettu

Ambu®-**silikoninaamari**

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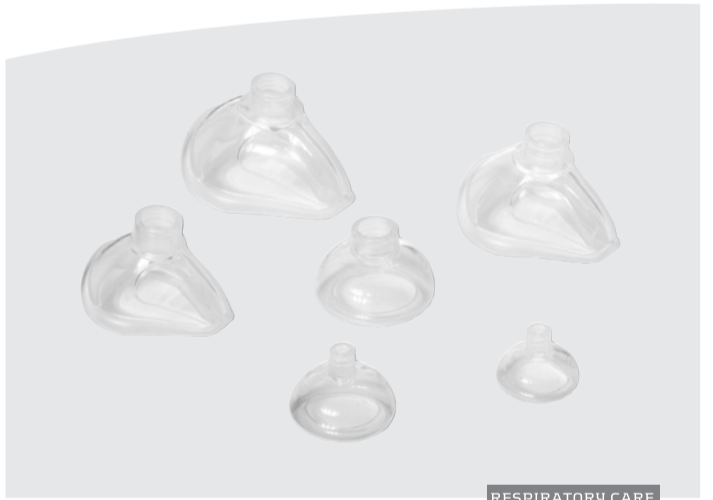
EN

Intended Use

The Ambu® Open Cuff Silicone Face Mask is a reusable face mask intended for use with manual resuscitators, automatic resuscitators and ventilators.

Instructions for Use

Remove mask from package and inspect. Apply the mask firmly to the patient’s face to achieve a tight seal. Hold the mask tight against the face while maintaining the correct head tilt to keep the airway open. Clean and sterilise the face mask according to instructions.

Materials:

Silicone Rubber

Cleaning and Sterilization - Reusable Item ONLY

To ensure safety of all kinds, please clean and sterilise

- Between patients,
- Whenever the product becomes contaminated,
- Every 24 hours of use with the same patient.

Carry out cleaning and sterilization as follows:

- Wash the components thoroughly in warm water containing a mild detergent. Ensure that the detergent is compatible with the component materials.
- Rinse the components thoroughly in warm water to remove all traces of detergent or fluid.
- Sterilise the components using one of the following methods:
 - Autoclave up to 134°C or 273°F.
 - Ethylene Oxide.
 - Most common disinfecting solutions that are used for equipment coming into contact with the patient.
- Rinse thoroughly with water after sterilization.
- Dry all components thoroughly after sterilization.
- Inspect all components for wear and tear or damage, and replace them as necessary. Then package appropriately to protect the device from soil and contamination.

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Ambu® is registered trademark of Ambu A/S